## FACE MASK (GB)

**Type IIR approved in accordance with EN 14683:2019**

**Intended use:** Prevention towards spread of infectious breath, cough and sneeze.

Protection against inhalation of airborne particles such as dust, pollen and transmittable bacteria and vira.

### Instruction for use:

1. Wash your hands thoroughly before AND after wearing.
2. Take hold of the mask at the upper edge, where the nasal clamp is felt and with the blue side facing outwards.
3. Place the nasal clamp over the nasal ridge and apply pressure to fasten the mask.
4. Holding the nasal clamp gently pull the lower edge of the mask downwards in order to expand the mask.
5. Place the earstrings firmly around ears.
6. Gently press the nasal bridge clamp around the nose to ensure a tight fit and adhesion.
7. Spread and unfold until it covers the chin completely, leaving no gaps between the mask cheeks and chin.
8. Dispose in closed bin

**Warning: Single use only. Do not use if the mask is dirty or broken. Keep out of reach of children. Non sterile.**

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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